


INSTRUCTIONS FOR USE
AIMOVIG® (AIM-oh-vig)
(erenumab-aooe)
injection, for subcutaneous use
140 mg/mL
single-dose prefilled SureClick® autoinjector

This Instructions for Use contains information on how to inject AIMOVIG with a SureClick autoinjector.

If your healthcare provider decides that you or a caregiver may be able to give your injections of AIMOVIG at home, you should receive training on the right way to prepare and inject AIMOVIG. Do not try to inject yourself until you have been shown the right way to give the injections by your healthcare provider or nurse.

The medicine in the AIMOVIG autoinjector is for injection under the skin (subcutaneous injection). See the AIMOVIG Patient Information for information about AIMOVIG.

Getting to know your prefilled autoinjector

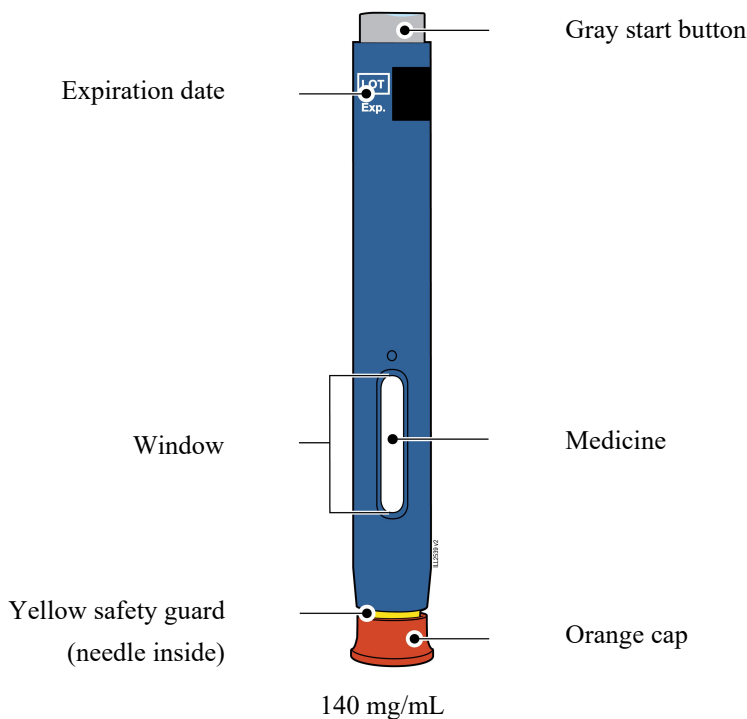

1. Important Information You Need to Know Before Injecting AIMOVIG

- It is important that you do not try to give the injection until you have fully read and understood this Instructions for Use.
- **Do not** use the autoinjector if the carton is damaged or the seal is broken.
- **Do not** use the autoinjector after the expiration date on the label.
- **Do not** shake the autoinjector.
- **Do not** remove the orange cap from the autoinjector until you are ready to inject.
- **Do not** use the autoinjector if it has been frozen.
- **Do not** use the autoinjector if it has been dropped on a hard surface. Part of the autoinjector may be broken even if you cannot see the break. Use a new autoinjector, and call 1-800-77-AMGEN (1-800-772-6436).
- The autoinjector is not made with natural rubber latex.

Frequently asked questions:


For additional information and answers to frequently asked questions, visit www.aimovig.com.

Where to get help:

If you want more information or help using AIMOVIG:

- Contact your healthcare provider,
- Visit www.aimovig.com, or
- Call 1-800-77-AMGEN (1-800-772-6436)

2. Storing and Preparing to Inject AIMOVIG

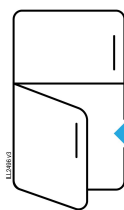


Refrigerate in carton
until ready to use

2a Refrigerate the autoinjector carton until you are ready to use it.

- Keep the autoinjector in the refrigerator between 36°F to 46°F (2°C to 8°C).
- Keep the autoinjector in the original carton to protect it from light or physical damage.
- **Do not** freeze the autoinjector.
- **Do not** store the autoinjector in extreme heat or cold. For example, avoid storing in your vehicle's glove box or trunk.

Important: Keep the autoinjector and all medicines out of the sight and reach of children.

WAIT

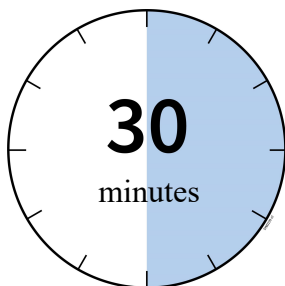

2b Wait 30 minutes for the autoinjector to reach room temperature.


- Let the autoinjector warm up naturally.
 - **Do not** heat the autoinjector with hot water, a microwave, or direct sunlight.
 - **Do not** shake the autoinjector at any time.
 - Using the autoinjector at room temperature makes sure the full dose is delivered and allows for a more comfortable injection.
-



2c You may keep AIMOVIG at room temperature for up to 7 days, if needed.

- For example, when you are traveling, you may keep AIMOVIG at room temperature.
 - Keep it at room temperature between 68°F to 77°F (20°C to 25°C).
 - **Do not** put it back in the refrigerator.
 - Record the date you removed it from the refrigerator and use it within 7 days.


Important: Place the autoinjector in a sharps disposal container if it has reached room temperature and has not been used within 7 days.

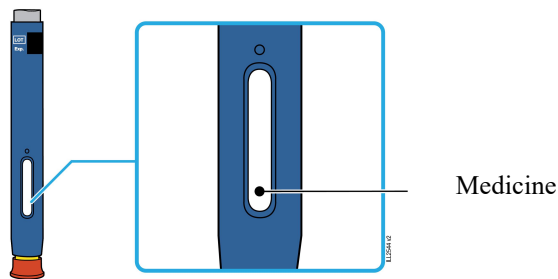

2d Inspect the medicine. It should be clear and colorless to slightly yellow.

- It is okay to see air bubbles in the autoinjector.
- **Do not** use AIMOVIG if the medicine is cloudy, discolored, or has flakes or particles.


Important: If the medicine is cloudy, discolored, or has flakes or particles, or if the autoinjector is damaged or expired, call 1-800-77-AMGEN (1-800-772-6436).

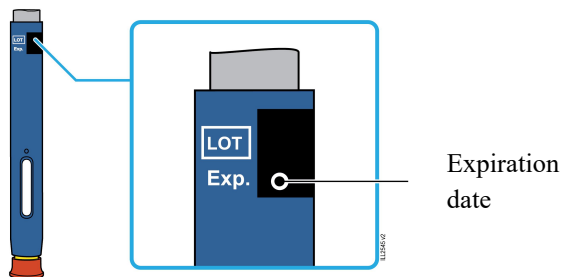

2e Check the expiration date (Exp.) and inspect the autoinjector for damage.


- **Do not** use the autoinjector if the expiration date has passed.
- **Do not** use the autoinjector if:
 - the orange cap is missing or loose.
 - it has cracks or broken parts.
 - it has been dropped on a hard surface.
- Make sure you have the right medicine and dose.

3. Getting Ready for Your Injection

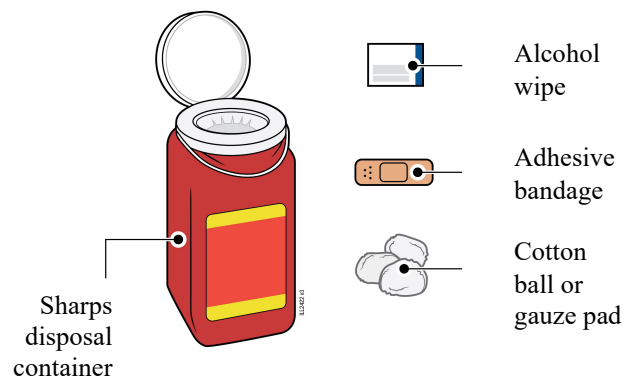

3a Gather and place the following items for your injection on a clean, flat, and well-lit surface:

- AIMOVIG autoinjector (room temperature)
- Sharps disposal container [see Disposing of AIMOVIG and Checking the Injection Site]
- Alcohol wipe
- Adhesive bandage
- Cotton ball or gauze pad

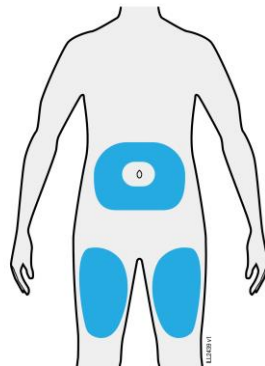


3b Inject into 1 of these sites.

- Inject into the front of your thigh or stomach (except for 2 inches around your belly button).
- Someone else can inject in your thigh, stomach, or back of the upper arm.

Important: Avoid areas with scars or stretch marks, or where the skin is tender, bruised, red, hard, raised, thick or scaly skin patch or lesion.



3c Wash hands thoroughly with soap and water.

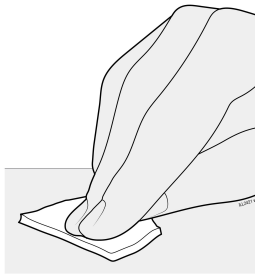


3d Clean the injection site with an alcohol wipe.

- Let the skin dry on its own.
 - **Do not** touch this area again before injecting.
-

4. Injecting AIMOVIG


Important: Only remove the orange cap when you can inject right away (within 5 minutes) because the medicine can dry out. **Do not** recap.

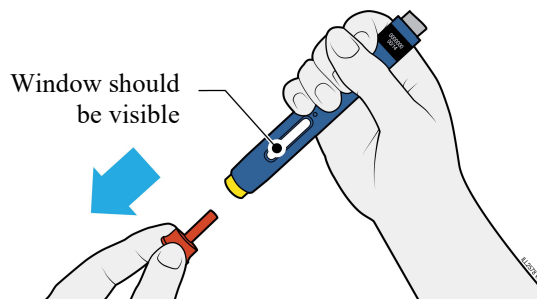

4a Grasp the autoinjector so you can see the window. Pull the orange cap straight off. You may need to pull hard.


- **Do not** twist, bend or wiggle the orange cap to pull it off.
 - **Never** put the orange cap back on. It may damage the needle.
 - **Do not** put your finger inside the yellow safety guard.
 - It is normal to see a drop of medicine at the end of the needle or yellow safety guard.
-

STRETCH

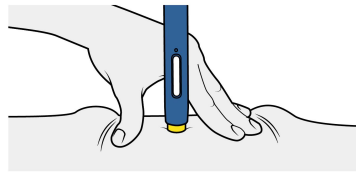

OR


PINCH

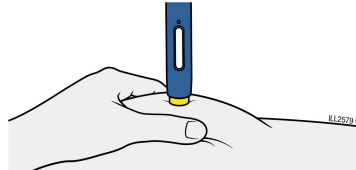

4b Stretch or pinch the skin to create a firm surface at the injection site. Place the yellow safety guard straight against the skin.

- Keep the skin **stretched or pinched** until the injection is finished.
- Make sure you can see the window.
- Make sure the autoinjector is positioned straight on the injection site (at a 90-degree angle).


PUSH
and hold against skin

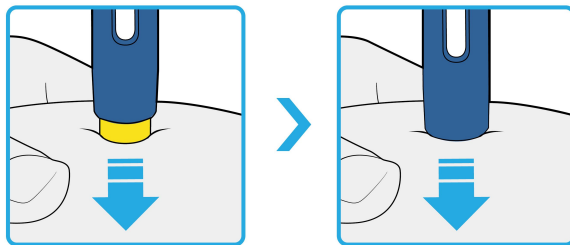

4c Firmly push the autoinjector down until the yellow safety guard stops moving. Hold the autoinjector down, do not lift.

- The yellow safety guard pushes in and unlocks the gray start button.


PRESS
gray start button

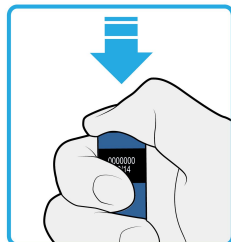

4d Keep pushing the autoinjector down and press the gray start button to start the injection.

- You may hear or feel a click.
- The window starts to turn yellow.
- It is okay to let go of the gray start button.

WATCH and CONFIRM
that the window turns fully yellow

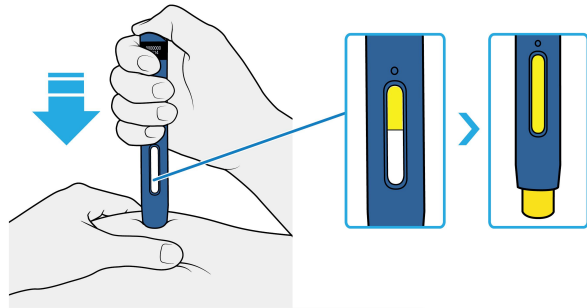

4e Keep pushing the autoinjector down. When the window is fully yellow, the injection is complete.

- The injection may take up to **15** seconds to complete.
- You may hear or feel a click.
- Lift the autoinjector away from your skin.
- The yellow safety guard locks around the needle.

Important: If the window has not turned fully yellow, or it looks like the medicine is still coming out, you have not received a full dose. Call your healthcare provider right away.


5. Disposing of AIMOVIG and Checking the Injection Site



5a Place the used autoinjector and orange cap in an FDA-cleared sharps disposal container right away after use.

Important: Do not throw away the autoinjector in your household trash.

- **Do not** reuse the autoinjector.
- **Do not** touch the yellow safety guard.

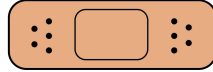

5b Check the injection site.

- **Do not** rub the injection site.
- If there is blood, press a cotton ball or gauze pad on your injection site. Apply an adhesive bandage if necessary.

Additional information about your sharps disposal container

If you do not have an FDA-cleared sharps disposal container, you may use a household container that is:

- made of a heavy-duty plastic,
- can be closed with a tight-fitting, puncture-resistant lid, without sharps being able to come out,
- upright and stable during use,
- leak-resistant, and
- properly labeled to warn of hazardous waste inside the container.

Disposing of sharps disposal containers:

When your sharps disposal container is almost full, you will need to follow your community guidelines for the right way to dispose of your sharps disposal container.

There may be state or local laws about how you should throw away used needles and syringes.

For more information about safe sharps disposal, and for specific information about sharps disposal in the state that you live in, go to the FDA's website at:

<http://www.fda.gov/safesharpsdisposal>

Do not dispose of your used sharps disposal container in your household trash unless your community guidelines permit this.

Do not recycle your used sharps disposal container.

Important: Keep the autoinjector and sharps disposal container out of the sight and reach of children.

For more information or help call 1-800-77-AMGEN (1-800-772-6436).

AIMOVIG[®] (AIM-oh-vig)

AMGEN

Manufactured by:

Amgen Inc.

One Amgen Center Drive

Thousand Oaks, California 91320-1799

U.S. License Number 1080

© 2019 - 2024 Amgen Inc. All rights reserved.

nnnnnnnnnn

This Instructions for Use has been approved by the U.S. Food and Drug Administration.

Revised: 8/2024
v10